

# LCD Electronic Digital Massager



**TO THE PATIENT:**

You have been diagnosed with muscle-related injuries. Such types of injuries has caused pain and discomfort. The electronic digital massager can help reduce such pain and discomfort by using micro-electronic current to stimulate blood circulation and using pulses to "exercise" injured muscle.

It is no longer necessary for many people whom have suffered muscle injuries or needs of exercise to endure muscle pains. The electronic digital massager can help to reduce the pain and discomfort.

**INDICATIONS:**

It is an electrically powered device intended for medical purposes, such as to relieve minor muscle aches and pains.

**WARNINGS:**

The safety use of electric digital massager must observe the following criteria:

1. It is not advice to use during pregnancy or during female monthly cycle.
2. The electrodes are not to be placed above the neck or near the head.

3. It is not to be used during driving, operating equipment or machinery.
4. People who are having a fever or flu.
5. Do not use during showering, bathing, or when sleeping.  
It is not recommended to use the electronic digital massager when using the following devices:
6. People who are using a pace-maker, or any implanted medical device.
7. People who have high blood pressure, heart disease.

#### PRECAUTIONS:

1. Isolated cases of skin irritations may occur at the area of electrodes placement following long-term application.
2. The effectiveness is highly dependent upon patient's selection of qualified physician or trainer in management of muscle rehabilitation.

#### ADVERSE REACTIONS:

Skin irritation and electrode burns (unattended pro-long use) can potential adverse reaction.

#### SPECIFICATION:

**Model No. : TS112201**

Channel: One

Power Source: 4.5V (1.5V "AA" alkaline batteries)

Pulse Width: Fixed at 100 $\mu$ s

Adjustable Intensity Levels: Eight

Six Massage Modes

Tapping I

Tapping II

Kneading

Pressing

Vibrating

Massager

Timer: 15 Minutes auto-shutoff

All values have  $\pm 10\%$  tolerance


#### USER'S INSTRUCTIONS:

Always read the instruction manual before use.

1. Installing the batteries:


Make sure that you are installing the batteries properly.

Note: Please install batteries according to their positive (+) and negative (-) ends correctly.

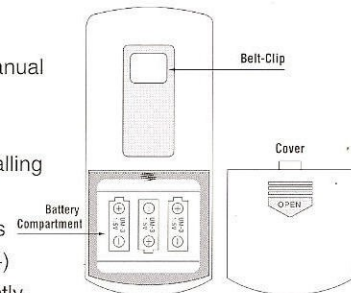


2. Connect the lead wires with "RED" and "WHITE" ends to the electrode pads.

Insert pins to the electrode sockets properly. Then insert the angled-end plug into the receptacle on the top of the main unit.

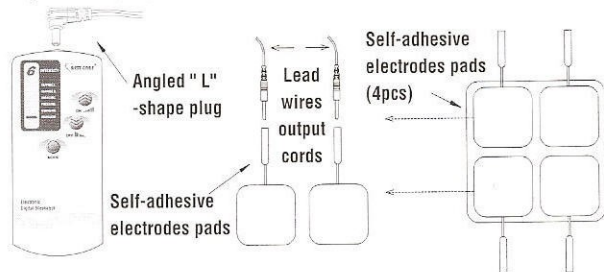


### 3. Applying Electrodes:

Before applying electrodes, be sure that the applied skin surface is thoroughly cleaned and dried. Place the electrodes over the skin, attach them properly, firmly and evenly.

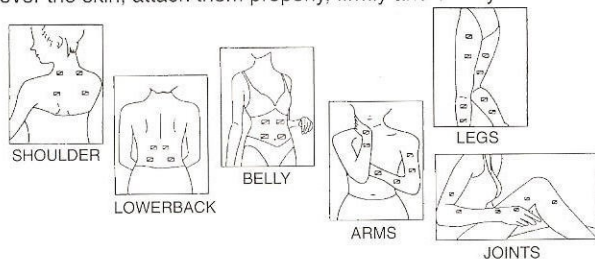

4. Switch on the main unit by pressing the blue button labeled "ON". Adjust the "INTENSITY" level to "1" (the lowest setting). When unsure what strength you will need, always start at the lowest intensity setting. Later, you may increase the intensity level as you desire.

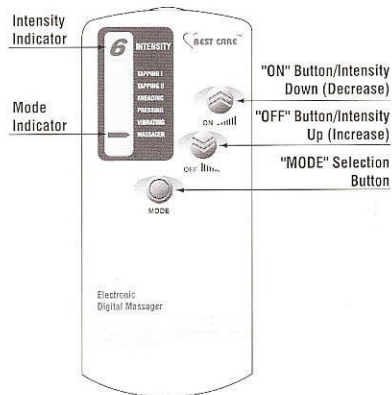

### ACCESSORIES

- SELF-ADHESIVE ELECTRODES 4 PCS.
- "AA" SIZE BATTERIES 3 PCS.
- LEAD WIRES 1 PC.
- INSTRUCTIONAL MANUAL 1 PC.
- HANDY CARRYING POUCH 1 PC.