

## Display Messages

**HI** *EZ Smart<sup>Plus</sup>* Meter detects blood glucose level is **higher** than 600 mg/dL (33.3 mmol/L).

**Lo** *EZ Smart<sup>Plus</sup>* Meter detects blood glucose level is **lower** than 20 mg/dL (1.1 mmol/L).

**.H** Ambient temperature was too high to perform the test.

**.L** Ambient temperature was too low to perform the test.

**.1** Test strip is not inserted properly, defective test strip, or insufficient blood sample.

**.3** Meter has some problems ; contact store/distributor.

□ **L-** Battery power is low. Meter will provide approximately 50 more measurements.

□ **.** Battery power is low. Replace immediately.

## Precautions for Use:

VIP International , Inc . cannot be held responsible for any consequences resulting from the incorrect use of the *EZ Smart<sup>Plus</sup> Blood Glucose Monitoring System* and not in line with the instructions set out in this operating manual.

**Only use *EZ Smart<sup>Plus</sup> Test Strip* with the *EZ Smart<sup>Plus</sup> Meter*.**

***EZ Smart<sup>Plus</sup>***  
*Blood Glucose Monitoring System*



## Quick Reference Guide

Distributed by VIP International, Wholesale Corp., Staten Island, NY 10304, USA

## Coding the meter

### STEP 1



Insert the *EZ Smart<sup>Plus</sup> Glucose Chip* into the slot. The *EZ Smart<sup>Plus</sup> Meter* will automatically be encoded.

### STEP 2



Match the code number on the *EZ Smart<sup>Plus</sup> Meter* display with the code number on the *EZ Smart<sup>Plus</sup> Test Strip* vial. If the code match, you may begin to test. If they do not match, follow Step 3.

### STEP 3

If the meter code and vial code do not match, please ask the store/distributor to replace a new vial.

## Test Summary

### STEP 1



Insert the Test Strip, white side up, contact bar end into the test port. A beep will sound and the code number will appear followed by a flashing "•" symbol. Now you are ready to obtain a blood sample.

### STEP 2



Place blood sample onto the test well of the *EZ Smart<sup>Plus</sup> Test Strip*. Wait until confirmation window has completely filled with blood.

### STEP 3



After 10 seconds, your blood glucose test result will appear on the screen and will be stored automatically.

## Memory Feature

### STEP 1



Press the button to turn on the *EZ Smart<sup>Plus</sup> Meter*. The 28 data average will automatically appear for few seconds then show the latest data.

### STEP 2



Each time you press the button, the *EZ Smart<sup>Plus</sup> Meter* will recall the last 28 test results accordingly. When the memory is full, the oldest result is removed and replaced with the newest result.

### STEP 3

To exit the Memory Mode and turn the Meter off. Press and release the button to the end of the memory or wait for 15 seconds.