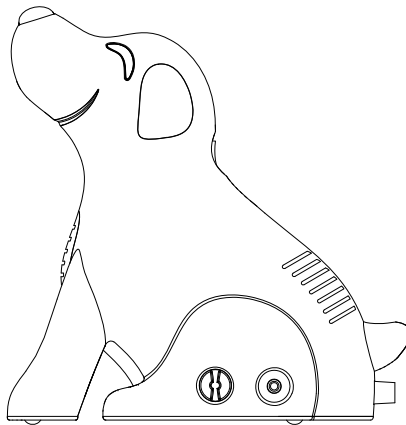
# Roscoe Dog

## COMPRESSOR NEBULIZER

### Guidebook

**MODEL: NEB-DOG**

**READ THIS INSTRUCTION  
MANUAL CAREFULLY  
BEFORE USE**

# Roscoe Dog COMPRESSOR NEBULIZER

MODEL NO.: NEB-DOG

## INSTRUCTIONS

### INDEX

1. Introduction	Page 2
2. Product Identification	Page 3
3. Important Safeguards	Page 3-4
4. Operating Your Compressor Nebulizer	Page 4-5
5. Cleaning	Page 5-6
6. Replacement Parts & Optional Accessories	Page 6
7. Specifications	Page 6-7
8. Warranty	Page 7

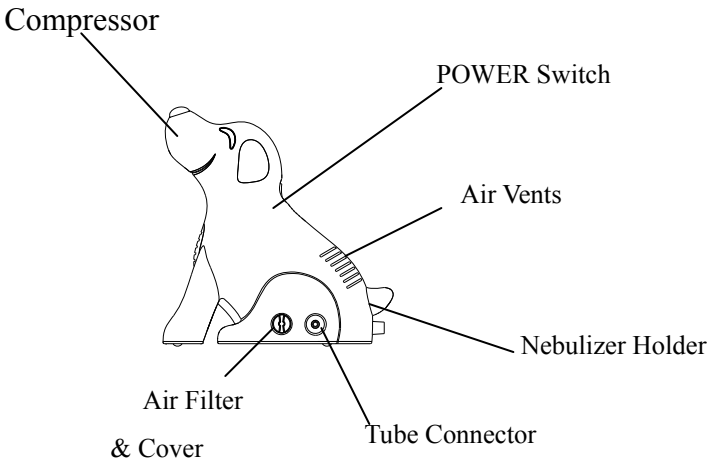
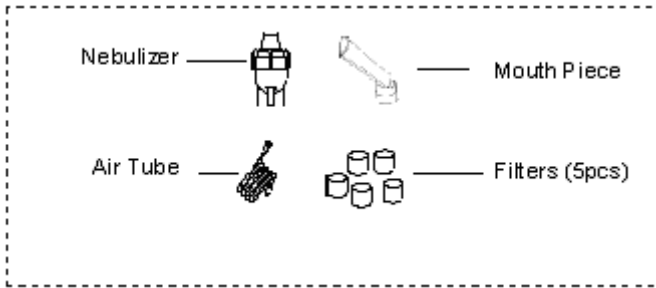
### 1. INTRODUCTION

Thank you for purchasing the Roscoe Dog Compressor Nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your compressor nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

## 2. PRODUCT IDENTIFICATION

## 3. IMPORTANT SAFEGUARDS

**Note:** Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product:

**Caution:** Failure to read and observe all precautions could result in personal injury or equipment damage.

### Product cautions:

1. To avoid electrical shock: Keep the unit away from water • Do not immerse the power cord or the unit in liquid • Do not use while bathing • Do not reach for a unit that has fallen into water – immediately unplug the unit.

2. Never operate the unit if it has any damaged parts (including power cord), if it has been dropped or submersed in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
5. Do not use other attachments unless recommended by manufacturer.

**Operating Cautions:**

1. Connect this product to an appropriate voltage outlet for your model.
2. Do not run this product unattended.
3. Never operate if this unit has a damaged cord or plug, if it has been dropped into water, or if it does not work properly. Return it to a service center for repair.
4. If any abnormality occurs, discontinue use immediately until the unit has been examined and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit or place it where the air openings may be obstructed.

**Storage Cautions:**

1. Do not store the unit under direct sunlight, high temperature or humidity.
2. Keep the unit out of reach of small children.
3. Always keep the unit unplugged while not in use.

**Cleaning Cautions:**

1. Do not immerse the unit in water. It may damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guidebook.

**4. OPERATING YOUR COMPRESSOR NEBULIZER**

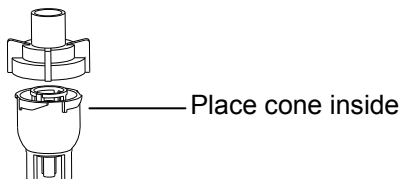
**Note:** Prior to initial operation, the nebulizer should be thoroughly cleaned.

1. Place your compressor nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.
2. Open the clear bag and remove the inside accessories.

**Important:** Prior to initial operation, the nebulizer should be thoroughly cleaned referring to the “Cleaning Procedures” in the manual.

3. Gently twist the top part of the nebulizer counter-clockwise to disassemble the nebulizer.

4. Fill the bottom section of the nebulizer with the medication prescribed by your physician. Be sure that the cone is put

inside the bottom section.

and fill medication

5. Gently twist the top part clockwise to re-assemble the nebulizer. Be sure the two sections fit well.
6. Attach one end of air tube to the base of the nebulizer.
7. Attach the other end of air tube to the air tube connector located on the side of the compressor.
8. Attach to the top section of nebulizer.
9. Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at "OFF" status.
10. Press power switch to begin your prescribed treatment.

**Important:**

The compressor motor has a thermal protector which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- a. Switch off the unit.
  - b. Unplug the unit from the electrical outlet.
  - c. Wait 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed.
11. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

## 5. CLEANING

It is recommended that the nebulizer and mouthpiece are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

### Rinsing (after each treatment)

1. Disconnect the air tube, nebulizer, and mouthpiece.
2. Gently twist the nebulizer to open it.
3. Rinse the nebulizer, mouthpiece with water.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

### Disinfections:

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece.
2. Complete Rinsing Steps 1-3.
3. Wash the nebulizer, mouthpiece in warm water and a mild detergent. Then wash them in hot tap water.
4. Submerge these parts in the vinegar and water solution for thirty minutes.

5. Complete Rinsing Steps 3-5.

**Cleaning the compressor**

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads, which may damage the finish.

**Filter change**

1. Do not use cotton or any other materials. Do not wash or clean the filter. Only use filters supplied by your distributor. And do not operate without a filter.
2. Change the filter every 30 days or when the filter turns gray.
3. Changing procedure:
  - A. Remove the filter cover.
  - B. Replace the used filter with a new one.
  - C. Replace the filter cover.

**6. REPLACEMENT PARTS & OPTIONAL ACCESSORIES**

Replacement parts are available through your local dealer.

**7. SPECIFICATIONS**

Power	AC 120V, 60Hz
Power Consumption	1.5 A
Medication Capacity	5ml
Particle Size	0.5 to 10 µm
MMAD	4 µm
Sound Level	Less than 60 dB at 1 meter
Average Nebulization Rate	≥0.2ml/min
Compressor Pressure Range	25 to 42 Psi (0.17 to 0.29 Mpa)
Operating Pressure Range	8 to 13 Psi (0.056 to 0.090 Mpa)
Liter Flow Range	5 ~ 8 lpm
Operating Temperature Range	10°C to 40°C (50°F to 104°F)
Operating Humidity Range	10 to 95% RH
Storage Temperature Range	-20°C to 70°C (-4°F to 158°F)
Storage Humidity Range	10 to 95 % RH
Dimension (L x W x H)	223.0 x 136.8 x 228.8 mm
Weight	3.5Lbs (1.6kg)
Standard Accessories	Nebulizer, Air Tube, Mouth Piece, Child Mask, Filters (5pcs)
Optional Accessories	Adult Mask, Extension Tube, T-adaptor

Protection against electric shock

 - Class II Equipment

 - Type BF applied parts: Mouth piece, masks.

Protection against harmful ingress of water

- IPX0 (Ordinary)

Degree of safety in the presence of flammable anesthetics or oxygen

- No AP/APG (Not suitable for use in the presence of flammable anesthetics or oxygen)

Mode of operation – 20 min on & 40 min off.

With respect to electric shock, fire and mechanical hazards only in accordance with UL 2601-1.

## **5 YEAR WARRANTY**

Your Roscoe brand product is warranted to be free of defects in materials and workmanship for five years.

This device was built to exacting standards and carefully inspected prior to shipment. Your Roscoe brand product is warranted to be free of defects in materials and workmanship. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Roscoe Medical.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to nondurable components, which are subject to normal wear and need periodic replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond Roscoe Medical's control, such as improper operation, maintenance or storage.

If you have any questions about your Roscoe product or this warranty, please contact an authorized Roscoe Dealer.

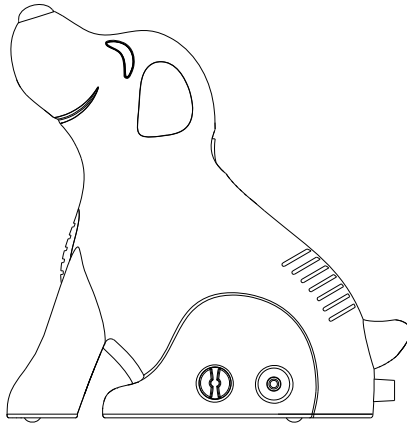
Manufactured for:  
**Roscoe Medical, Inc.**  
Strongsville, Oh 44149

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